


GEISTER

GEISTER®

The better way to operate™

PMOD


Peng's Multifunctional
Operational Dissector

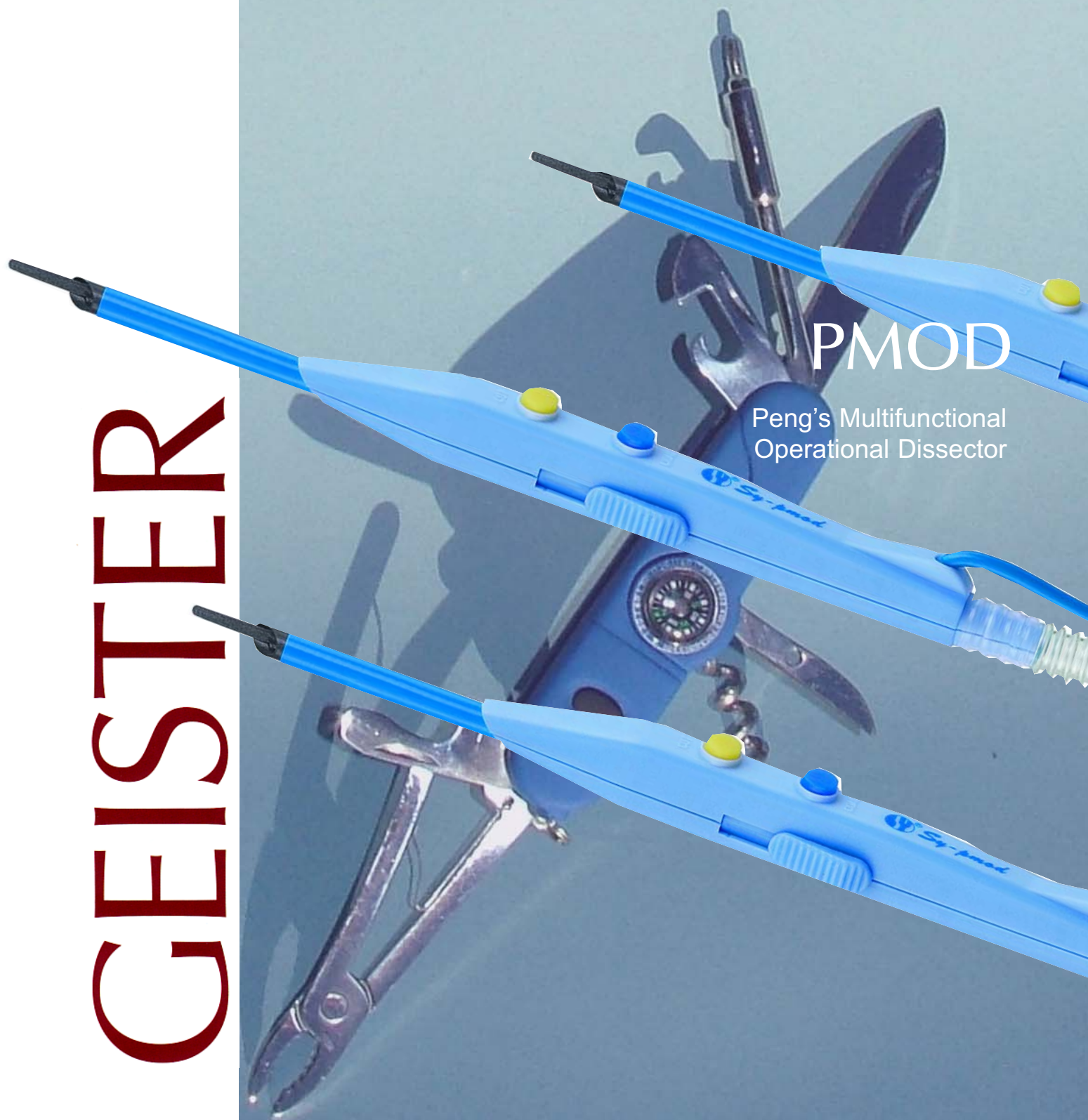

Peng's Multifunktional Operational Dissector (PMOD)

One of the most efficient, economic and convenient instruments applied in surgical dissection so far, the PMOD has been widely recognized and adopted by surgeons worldwide. Uniquely designed, it combines the 4 functions cutting, separating (curetting), aspirating and coagulating. The system has a metal tube whose body is insulated except a bevel edge at the distal end. A movable blade is attached at the tip for diathermy cutting. The suction pipe is connected with the suction apparatus through the aspiration tube and the monopolar cable is connected to the electrosurgical unit.


With the exception of suturing, the PMOD allows the operator to carry out all operative manipulations including cutting, haemostasis suction and dissections. By eliminating the need to frequently change surgical instruments during operations, the PMOD saves operation times, reduces haemorrhage, greatly improves handling and quality of operation and reduces medical expenses. Using the PMOD, harmful smoke produced during electro-coagulation can be sucked out, to the benefit of the health of the medical staff and environment.

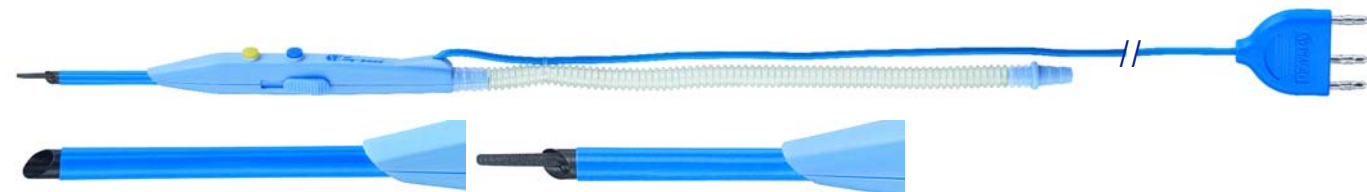

34-9450 IIB-1 Single-use Dissector Handle, regular shaft 40-60 mm expandable with pushing key, blunt tip, dissecting edge, with suction/smoke evacuation, non-stick coating, 2 push-button switches, 2.8 m cable with US 3-pin plug




34-9452 IIB-2 Single-use Dissector Handle, long shaft 90-110 mm expandable with pushing key, blunt tip, dissecting edge, with suction/smoke evacuation, non-stick coating, 2 push-button switches, 2.8 m cable with US 3-pin plug

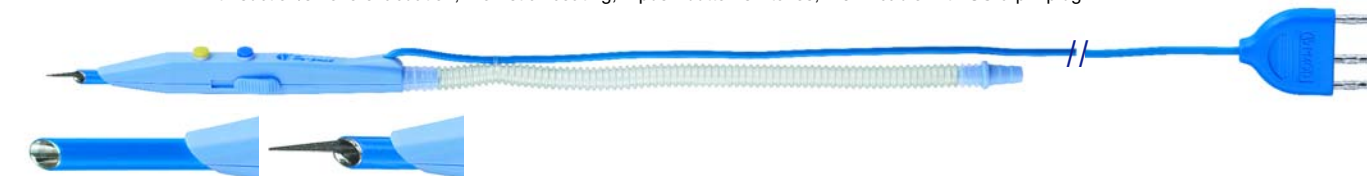

34-9454 IIB-3 Single-use Dissector Handle, short shaft 10-30 mm expandable with pushing key, pointed tip, with suction/smoke evacuation, non-stick coating, 2 push-button switches, 2.8 m cable with US 3-pin plug (recommended for thyroidectomy, mastectomy etc.)




34-9460 IIIA-1 Single-use Dissector Handle, blunt tip, plain, 2 push-button switches, 2.8 m cable with US 3-pin plug

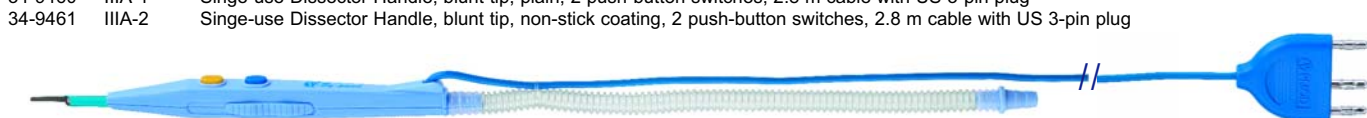

34-9461 IIIA-2 Single-use Dissector Handle, blunt tip, non-stick coating, 2 push-button switches, 2.8 m cable with US 3-pin plug



34-9470 IIIX-1 Single-use Dissector Handle, blunt tip, with suction/smoke evacuation tube movable with pushing key, plain, 2 push-button switches, 2.8 m cable with US 3-pin plug

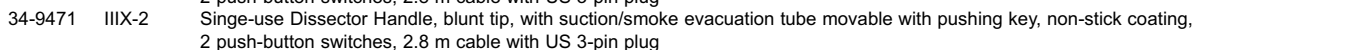

34-9471 IIIX-2 Single-use Dissector Handle, blunt tip, with suction/smoke evacuation tube movable with pushing key, non-stick coating, 2 push-button switches, 2.8 m cable with US 3-pin plug

34-9480 Adapter for US 3-pin plug to connect to Erbe ACC/ICC generators

34-9481 Adapter for US 3-pin plug to connect to Erbe T-series generators

34-9482 Adapter for US 3-pin plug to connect to Martin, Berchtold generators



The better way to operate™

Geister Medizintechnik GmbH
 Föhrenstraße 2
 D-78532 Tuttlingen/Germany
 Telefon +49 (0) 74 61 - 96 62 40
 Telefax +49 (0) 74 61 - 96 62 422
 E-mail info@geister.com
 Internet www.geister.com

Distributor