



GEISTER

GEISTER®
The better way to operate™

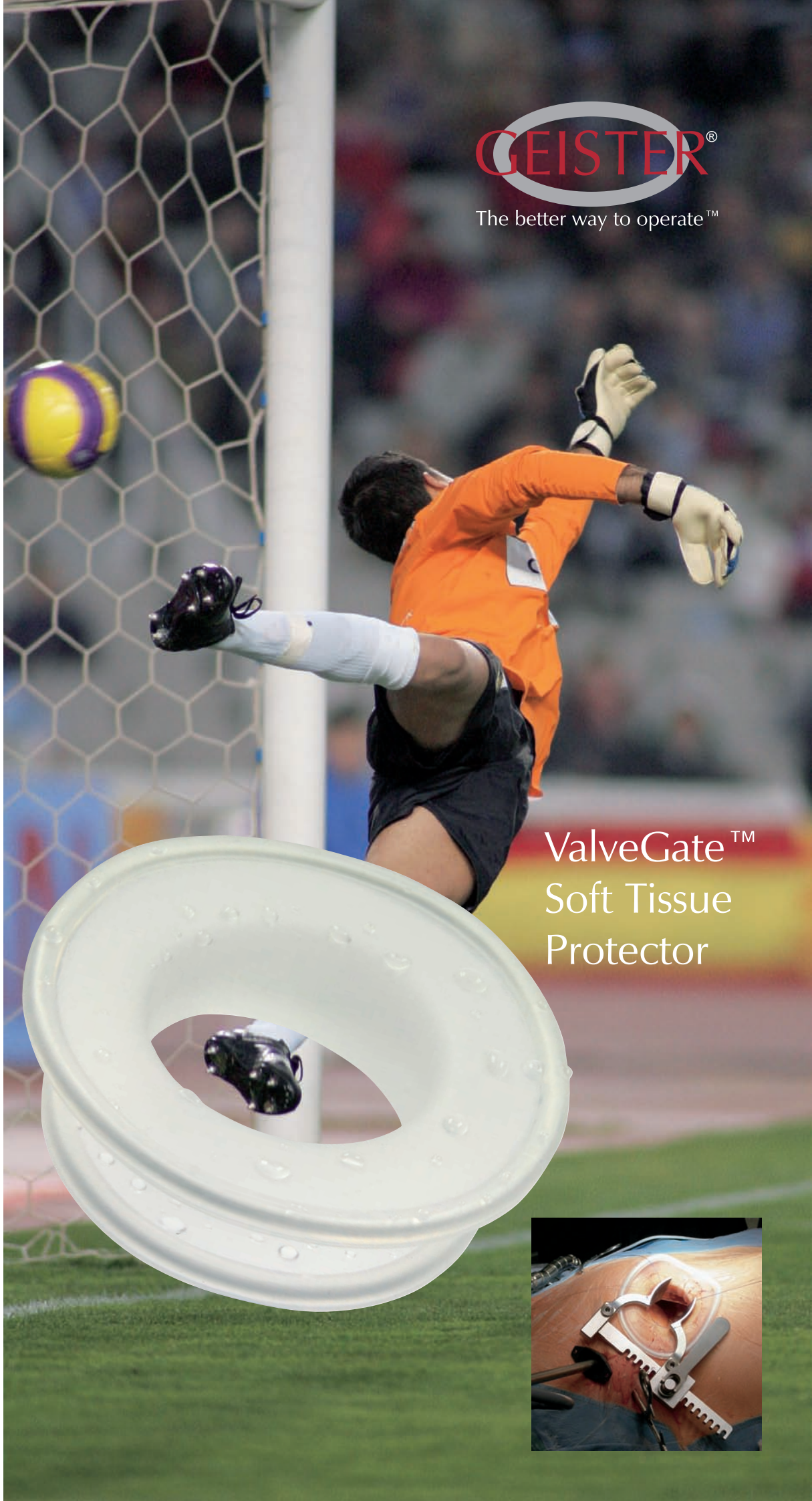

ValveGate™
Soft Tissue
Protector


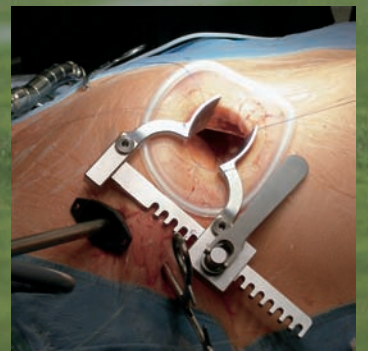


ValveGate™ Soft Tissue Protector

- for minimally invasive valve surgery and V.A.T.S.
- protects wound edge and surrounding tissue
- improves visibility and exposure
- very easy to insert in situ
- high flexibility
- self-holding, for incisions from 6 to 9cm (regular size) and from 3 to 5cm (small size)
- latex-free silicone rubber

Order information:
ValveGate™ Soft Tissue Protector,
sterile, single use, pack of 2

Regular Size REF 29-1399
Small Size REF 29-1399S

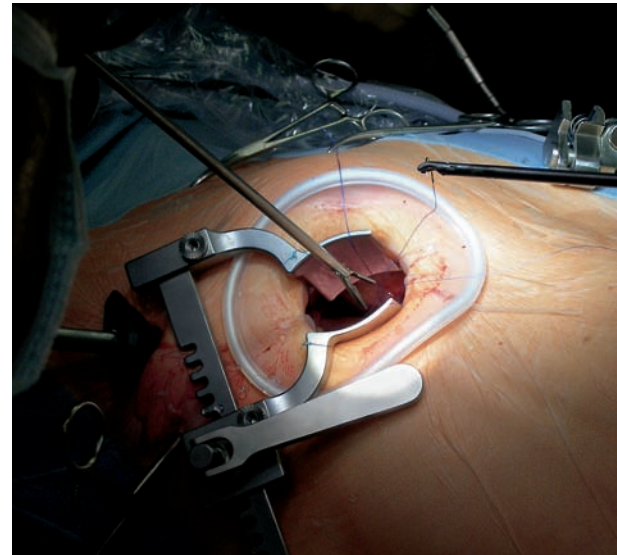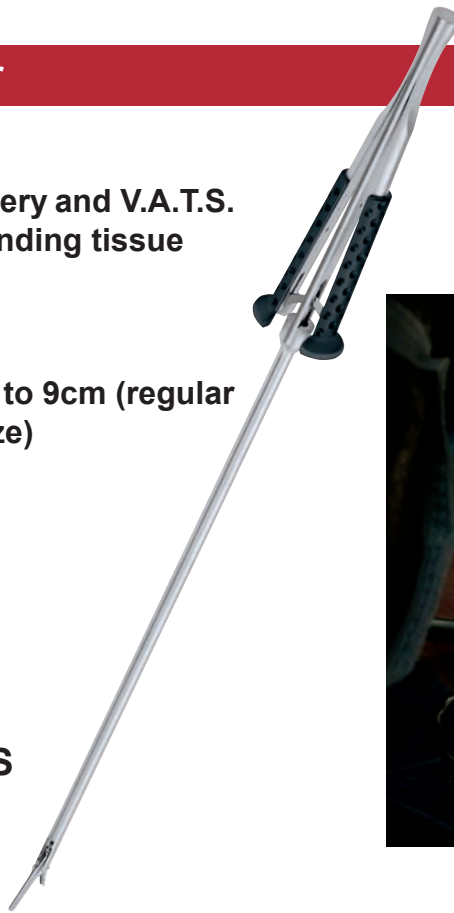

The ValveGate™ Soft Tissue Protector for Minimally Invasive Valve Surgery is a sterile, single patient use wall access device consisting of 2 overlaid plastic rings that are interconnected by means of a silicone rubber membrane. The bottom ring is a flexible ring made with a shape memory alloy to be inserted in situ.

For easy placement simply push the flexible ring downwards as illustrated and insert it into the wound cavity. Then push the reversed portion carefully down as well and expand it.

Remove it in the opposite way by inserting some fingers, grabbing a part of the flexible ring, and pulling it out carefully.

Dispose after use.

© Geister 2010 · Printed in Germany · All rights reserved.



The better way to operate™

Geister Medizintechnik GmbH
Föhrenstrasse 2
D-78532 Tuttlingen/Germany
Phone +49 (0) 7461 – 966 240
Telefax +49 (0) 7461 – 966 2422
E-mail info@geister.com
Internet www.geister.com

Distributor